

# EASYRECLINER<sup>HEALTH</sup> AGED CARE



## OPERATING INSTRUCTIONS

To operate the recline mechanism push forward on both arms. This action will release the mechanism and allow the seat to slide forward – at the same time by leaning backwards against the backrest the mechanism will continue its reclining action.

At the same time the leg support is activated.

Note the spring activated and counter balanced mechanism can be set in any position by ceasing to apply pressure to leg support or pushing on the arm.

To return the seat, back and leg support to the closed or upright position apply downward pressure with legs to the leg support while sitting forward.

### Special notes:

- Ensure the castors if fitted are not locked before attempting to relocate the chair.
- This product is not designed to travel over gaps in floors such as at lift entrances.
- Do not allow persons to sit on the backrest when in the reclined position as permanent damage to the mechanism will occur.
- This product is not intend as a patient transfer system.

## L - ERLNG1

- Generous seat
- Superbly balanced push arm reclining action
- Leggett & Platt Mechanism tested to 25000 loaded cycles at 120kg
- Fixed seat & back cushion
- Non marking hospital grade static front & lockable rear swivel castor
- Reclining mechanism for patient use or for patient transfer
- Mobility back handles for staff use
- Optional meal tray

Size: 1110H x 780W x 880D  
 Recline depth: 1660mm  
 Seat height: 520mm  
 Arm Height: 680mm

Options: C - Castors  
 DT - Meal tray  
 H - Handles

Sturdy Framac pursue a policy of continuous improvement and reserve the right to introduce modification when required and to withdraw models without previous notification.

Sturdy Components Pty Limited trading as  
**STURDY FRAMAC**  
[www.sturdy.com.au](http://www.sturdy.com.au)

95 Gow Street  
 Padstow NSW 2211  
 Australia  
 Telephone: (02) 8760 8300  
 Fax: (02) 9707 1081

[sales@sturdy.com.au](mailto:sales@sturdy.com.au)

